

HCV Ag Only Seroconversion Panel

REF Catalogue No: SCP-HCV(Ag)-007

The Biomex GmbH HCV Ag Seroconversion Panel consists of 18 members with each member containing 1.0 mL of human plasma. This panel illustrates the onset of the Hepatitis C virus (Genotype 3a) core antigen from one individual over a period of 113 days.

1. Intended Use

This Seroconversion Panel (SCP) is intended for standard testing by diagnostic manufacturers and researchers during assay development, evaluation, troubleshooting and post-marked surveillance of Hepatitis C core antigen and Ag/Ab test systems and methods. Moreover, it serves as validation tool for diagnostic sensitivity, determination of analytical sensitivity or the identification of cut-off values.

2. Storage and Stability

Store the SCP at $< -20^{\circ}\text{C}$ to -80°C . Thaw samples at room temperature and mix gently by inversion before usage. Avoid foaming, contamination and repeated freeze and thaw cycles.

3. Warnings and Precautions

Infectious material. This product is capable of transmitting infectious diseases. Do not pipette by mouth.

Prevention:

P264: Wash hands thoroughly after handling.

P270: Do not eat, drink or smoke when using this product

P273: Avoid the release to the environment

Disposal:

P501: Dispose of waste in accordance to applicable local or national regulations. Waste must be disposed in a secured manner.

4. Donor Information

All panel members have been tested and found negative/non-reactive for anti-HIV 1/2, anti-HCV and HBsAg by CE marked tests. Furthermore the members were tested non-reactive for HIV RNA and HBV DNA.

5. Detection Methods

Each panel member is assayed for HCV Ag with Abbott ARCHITECT[®], anti-HCV with Abbott ARCHITECT[®], Siemens ENZYGNOST[®] and Roche COBAS[®] e411, HCV Ag/Ab with DiaSorin MUREX[®] and Bio-Rad MONOLISA[®], as well as qualitative testing is performed for the direct detection of HCV RNA with Roche COBAS[®] MPX 6800/8800. Viral load determination is performed with the Abbott m2000 System. Testing is carried out with CE marked tests.

6. Limitations and Restrictions

This panel is for Research Use Only and not intended for human or animal diagnostics, or for therapeutic purposes. Each laboratory has the responsibility to ascertain the suitability of the SCP its particular application and to establish their own guidelines for interpretation of results. Data is provided for informational purposes only. The Biomex GmbH does not claim that others can duplicate these test results exactly.



Declaration of used symbols

Catalog Number	Lot Number	Consult Instructions for Use	Manufactured by	Temperature Limitation

HCV Ag Only Seroconversion Panel, Catalogue No: SCP-HCV(Ag)-007

Panel member	Donor	Lot#	Day	Bleed Date	Anti-HCV Abbott Architect S/CO	HCV Ag Abbott Architect fmol/L	Roche Cobas e411 HCV Ab COI	Siemens Enzygnost Anti-HCV 4.0 S/CO	DiaSorin Murex HCV Ag/Ab S/CO	Bio-Rad Monolisa HCV Ag-Ab Ultra Assay S/CO	HCV MPX Qualitativ Roche Cobas 6800/8800	HCV viral load Abbott m2000 System IU/mL
					cut off ≥ 1 s/co	cut off ≥ 3 fmol/L	cut off ≥ 1 COI	cut off ≥ 1 s/co	cut off ≥ 1 s/co	cut off ≥ 1 s/co		
1	524693959	360101564	0	16.07.2012	0.05	0.98	0.07	0.01	0.39	0.28	Non Reactive	<12
2	524693959	360106670	36	21.08.2012	0.05	0.22	0.07	0.02	0.37	0.26	Non Reactive	<12
3	524693959	360107267	39	24.08.2012	0.05	0.16	0.08	0.01	0.36	0.25	Non Reactive	<12
4	524693959	360107666	43	28.08.2012	0.04	0.43	0.07	0.01	0.35	0.28	Non Reactive	<12
5	524693959	360108344	46	31.08.2012	0.05	0.26	0.07	0.01	0.38	0.26	Non Reactive	<12
6	524693959	360109243	53	07.09.2012	0.05	0.32	0.07	0.01	0.38	0.27	Non Reactive	<12
7	524693959	360109626	57	11.09.2012	0.04	0.31	0.07	0.02	0.61	0.22	Non Reactive	<12
8	524693959	360110187	60	14.09.2012	0.04	0.04	0.07	0.02	0.44	0.20	Non Reactive	<12
9	524693959	360110658	64	18.09.2012	0.04	0.00	0.07	0.02	0.49	0.20	Non Reactive	<12
10	524693959	360111177	67	21.09.2012	0.04	0.00	0.07	0.01	0.49	0.20	Non Reactive	<12
11	524693959	360111585	71	25.09.2012	0.04	0.00	0.07	0.01	0.52	0.19	Non Reactive	<12
12	524693959	360112237	74	28.09.2012	0.04	0.00	0.07	0.01	0.45	0.21	Non Reactive	<12
13	524693959	360114120	88	12.10.2012	0.05	82.29	0.07	0.02	0.38	0.21	Reactive	24724
14	524693959	360114461	91	15.10.2012	0.06	4471.16	0.07	0.02	0.84	0.25	Reactive	1048441
15	524693959	360116025	101	25.10.2012	0.04	17605.32	0.07	0.02	1.81	0.44	Reactive	4069837
16	524693959	360116639	106	30.10.2012	0.05	15036.30	0.07	0.02	2.43	0.68	Reactive	4813347
17	524693959	360117013	108	01.11.2012	0.05	16443.75	0.07	0.02	3.37	0.69	Reactive	8905073
18	524693959	360117573	113	06.11.2012	0.04	39679.15	0.07	0.01	4.86	1.08	Reactive	4036100

HCV Ag Only Seroconversion Panel, Catalogue No: SCP-HCV(Ag)-007

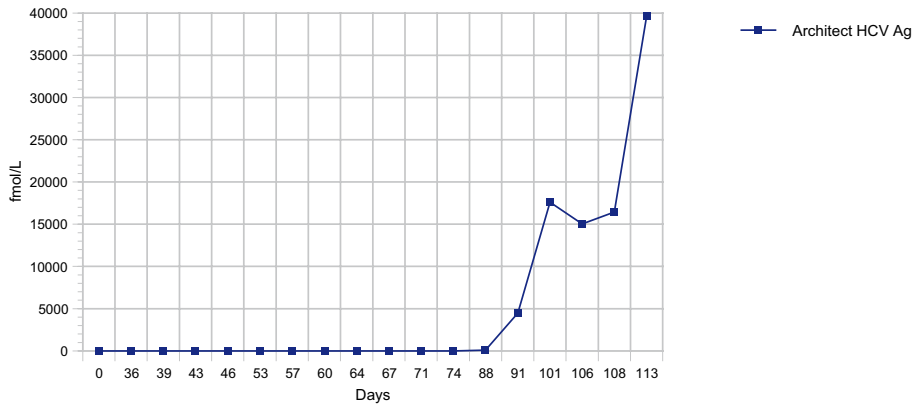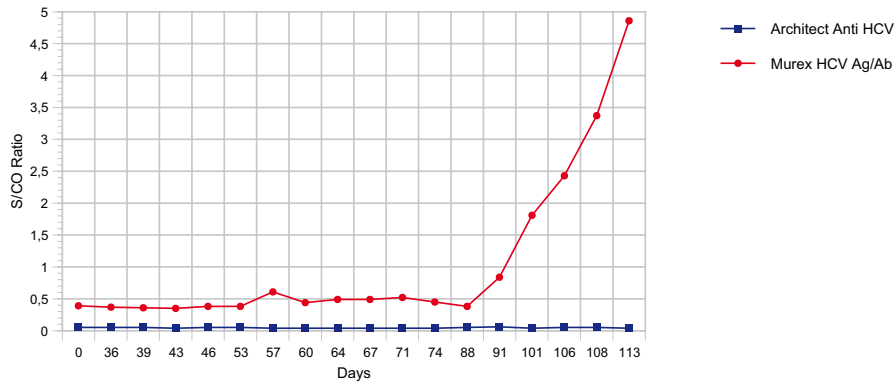

Assay information:

Assay	Lot#	Exp. Date	Test Date
Abbott Architect Anti-HCV assay	62999LI00	13.10.2016	31.03.2016 01.04.2016
Abbott Architect HCV Ag assay	59161LP39	08.10.2016	25.04.2016
Roche Anti-HCV assay	131785-01	01.09.2016	26.04.2016
Siemens Enzygnost Anti-HCV 4.0	45981	09.12.2016	02.05.2016
DiaSorin Murex HCV Ag/Ab	C931010	28.02.2017	28.04.2016
Bio-Rad Monolisa HCV Ag-Ab Ultra assay	5J0022	30.08.2016	06.06.2016
Abbott Architect HIV Ag/Ab combo assay	62998LI00	22.10.2016	31.03.2016 01.04.2016
Abbott Architect HBsAg qualitativ II assay	61151FN00	25.01.2017	31.03.2016 01.04.2016
Roche cobas® MPX assay	209572	31.10.2016	18.04.2016
Abbott RealTime HCV Amplification kit	466744	15.08.2017	14.11.2016